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Lebende Zahnformenkarte
Carte de formes vivantes
Carta de formas en vivo

Working mould guide
Tabella forme in-vivo
Levende Vormenkaart

Aesthetic Basismaterial

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Introduction

Dear customer,

The product that you have purchased qualifies as a medical device under European Directive 93/42/EEC. Please ensure that you file the lot number and product name in your administrative system on receipt of the product. For each piece of work that you produce, please make a note of all the materials used along with all the appropriate lot numbers. For your own protection as well as for the protection of your patients and the environment, please also follow the important guidelines in the following Instructions for Use:

Product description

C-Plast cold curing tray material is ideal for the production of custom trays, base plates and registration plates. To obtain the best results, please follow the instructions contained in this leaflet.

White: C-Plast is a cold curing precision tray material for the production of registration plates, custom trays and precision base plates for bite rims.

Colour 34: C-Plast is a cold curing precision tray material which is particularly well suited to the production of base plates for trying in full dentures.

Composition	(% weight)
Polymer	
Polymethyl methacrylate (CAS No. 9003-42-3)	25.4 %
Copolymer consisting of ethyl and methyl methacrylate polyethyl methacrylate (CAS No. 9011-14-7)	< 50.0 %
Calcium carbonate	24.5 %
Catalyst (benzoyl peroxide) (CAS No. 94-36-0)	< 1.0 %
Pigment (yellow pigment in pink C-Plast only)	0–0.3 %
Monomer	
Methyl methacrylate (CAS No. 80-62-6)	< 90.0 %
Paraffin oil	10.0 %
Diethanol-p-toluidine (CAS No. 3077-12-1)	0.5 %

Safety data sheets can be found on our website at www.candulor.com

Benefits

- Good contrast with most impression materials
- Enhanced dimensional stability
- Simple dosing system
- Does not smear (good grinding characteristics)
- Excellent consistency

Storage

- Permitted ambient temperature 12 °C to max. 28 °C (54 °F to max 83 °F)
- Protect from light
- Do not use products after the stated expiry date.
- **Store the monomer container in a dark, cool and well-ventilated place**

Indication

- Custom trays
- Custom base plates for:
 - bite rims
 - intraoral registration plates
 - setting up teeth with wax
- Production of custom incisal guiding plates

Contraindication

- Do not apply C-Plast in the oral cavity
- Not suitable for alginates and hydrocolloids
- Do not use for denture bases

Side effects

No systemic side effects have been reported to date. In the event of an allergic reaction to C-Plast ingredients, stop using the product immediately.

Warning notice

- Monomer contains methyl methacrylate (MMA)
- MMA is an irritant and easily flammable (flash point: + 10 °C / 50 °F)
- MMA and its vapours are irritating to the eyes, skin and respiratory system
- May cause sensitisation upon skin contact
- Do not inhale vapours
- Keep away from sources of ignition – no smoking
- Do not empty into drains
- Avoid contact of the skin, mucosa and eyes with monomer and uncured material. Many commercial gloves, e.g. those made of latex or vinyl, are not monomer-resistant and therefore do not provide effective protection against the sensitising effect of methacrylates
- Wear a mask when grinding and use a suction removal system
- Always use a spatula when handling the mixture

Instructions for Use

1. Preparation

The plaster surfaces must be well wetted. Isolate the surfaces with two applications of Candulor ISO-K and allow to dry thoroughly.

2. Dosage

Mixing ratio for one complete tray:

C-Plast polymer: 1 level scoopful / 21 g

C-Plast monomer: 6 ml in the measuring cylinder

Blend the powder and liquid mixture thoroughly in the mixing cup using a spatula.

3. Working time

As soon as the material no longer sticks, it can be processed for approx. 5 minutes at 23 °C / 73 °F. The higher the room temperature, the shorter the working time.

The dough can be used directly on the model or placed between two polyethylene sheets and rolled out to form an even layer (ideal thickness 3–4 mm).

Shape the rolled out dough to fit the isolated plaster model, using a pair of scissors or scalpel to remove the excess.

4. Polymerisation

The polymerisation time is approx. 7 minutes

(at 23 °C / 73 °F) after mixing.

Lift the polymerised pieces from the model. Trim the edges.

Repairs and corrections

Repairs or corrections may be carried out using C-Plast. The contact surfaces must be well roughened with a bur and wetted with monomer.

Important

This material has been developed solely for use in dentistry and must be handled strictly in accordance with the Instructions for Use. The manufacturer cannot be held liable for any damages arising as a result of failure to observe the Instructions for Use or the stated area of application. The same applies in the event that the product is mixed or processed with other manufacturers' products. The user shall be solely responsible for testing the material with respect to its suitability prior to use for any purpose other than those explicitly stated in the instructions.

CAUTION: US Federal law restricts this device to sale by or on the order of a licensed dentist.

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Art.-Nr. 90300070 01/2011 3000 REV02 www.teama5.com © Copyright 2011 Candulor AG, Wangen/ZH