Iso-K

Isolierflüssigkeit für alle Kunststoffe
Separating liquid for all types of acrylic
Liquide isolant toutes résines
Liquido isolante per tutte le resine
Isolatievloeistof voor alle kunstharsen
Separador para todo tipo de resinas
Isoleringsvæske for alle plasttyper

Verarbeitungsanleitung
Instructions for Use
Instructions de mise en œuvre
Istruzioni d'uso
Gebruiksaanwijzing
Instrucciones de uso
Forarbejdningsvejledning

Candolor
Kunst Zahn Werk

Deutsch

ISO-K

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English

ISO-K

Separating liquid for all types of acrylic

Introduction

Thank you for purchasing Iso-K. Candulor Iso-K is an alginate-based separating liquid for isolating acrylics against plaster. It can also be used to separate the two halves of the flask when filled. When used correctly, Iso-K will make work much easier and will leave the surface of the polymerised acrylic smooth and plaster-free.

Instructions for Use

Proper wax removal and well wetted plaster surfaces are essential for the successful use of this product. To remove any traces of wax and grease it is advisable to boil out the model using water only, i.e. without washing-up liquid.

Important:

In the case of acrylic teeth, isolate before drilling retentions.

Options

1. Iso-K can be applied to lukewarm or cold models. Always ensure that the first layer applied has hardened before applying the second layer. To obtain the best results, apply two thin coats of the separating liquid.
2. Pour some Iso-K into a small receptacle and apply it with a soft-haired brush which has been slightly dampened beforehand. The brush must be rinsed thoroughly in running water after use.
Iso-K can also be applied with a spray gun.

Note

A slight sediment may form if the bottle is stored for a longer period of time. This will not affect the isolating capability of the liquid and can be prevented by simply shaking the bottle before use.

Storage

Permitted ambient temperature 18 °C to 25 °C
(65 °F to 77 °F)

Shelf life

3 years, subject to correct storage and stated expiry date - please read the label!

Important

This material has been developed solely for use in dentistry and must be handled strictly in accordance with the Instructions for Use. The manufacturer cannot be held liable for any damages arising as a result of failure to observe the Instructions for Use or the stated area of application. The same applies in the event that the product is mixed or processed with other manufacturers' products. The user shall be solely responsible for testing the material with respect to its suitability prior to use for any purpose other than those explicitly stated in the instructions.

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Candulor AG
Subsidiary: Candulor Dental GmbH
Subsidiary: Candulor USA Inc.